



Sample test #1

Paracheck Pf Device Rapid Test for *P. falciparum* Malaria (Ver.3) (30301025)

1

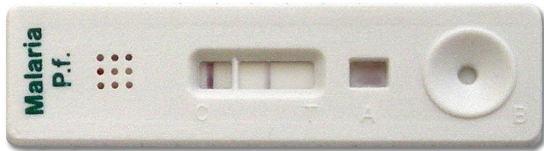

6

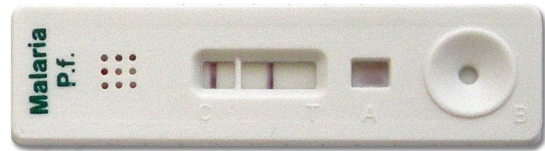

2



7




3




8




4

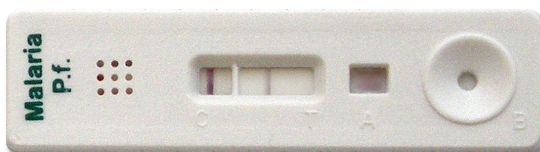

9

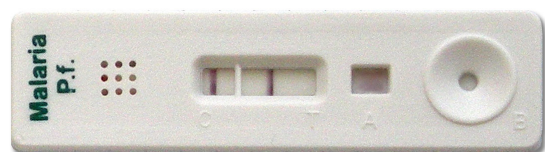

5

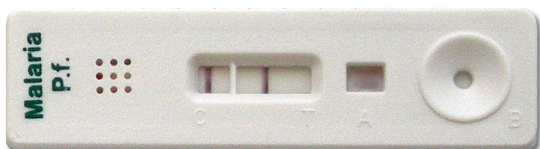

10



Sample test #2

Paracheck Pf Device Rapid Test for *P. falciparum* Malaria (Ver.3) (30301025)

1



6



2




7



3



8

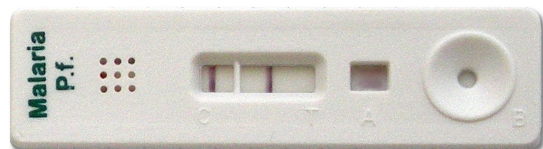


4



9



5

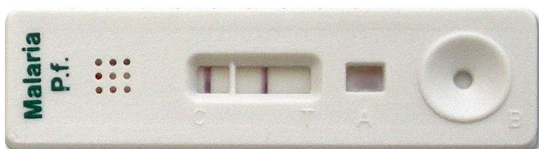

10



Sample test #3

Paracheck Pf Device Rapid Test for *P. falciparum* Malaria (Ver.3) (30301025)

1



6



2




7



3



8

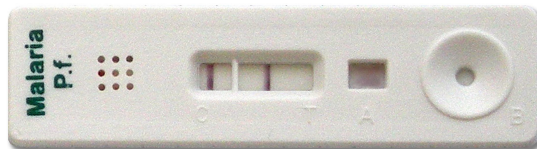


4



9



5

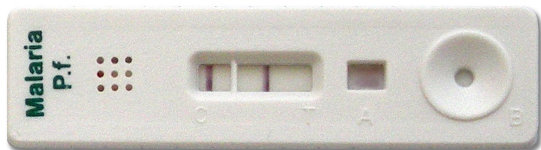

10



Answer keys for sample tests

Sample test #1 Paracheck Pf Device Malaria (Ver.3) (30301025)

	Positive (+)	Negative (-)	Invalid
1.	✓		
2.		✓	
3.		✓	
4.	✓		
5.	✓		
6.	✓		
7.		✓	
8.			✓
9.	✓		
10.	✓		

Sample test #2 Paracheck Pf Device Malaria (Ver.3) (30301025)

	Positive (+)	Negative (-)	Invalid
1.		✓	
2.	✓		
3.		✓	
4.			✓
5.	✓		
6.	✓		
7.			✓
8.	✓		
9.	✓		
10.		✓	

Sample test #3 Paracheck Pf Device Malaria (Ver.3) (30301025)

	Positive (+)	Negative (-)	Invalid
1.			✓
2.		✓	
3.	✓		
4.	✓		
5.	✓		
6.		✓	
7.	✓		
8.	✓		
9.	✓		
10.			✓

Sample test #1

1. Positive
2. Negative
3. Negative
4. Positive
5. Positive
6. Positive
7. Negative
8. Invalid (No control)
9. Positive
10. Positive

Sample test #2

1. Negative
2. Positive
3. Negative
4. Invalid (No control)
5. Positive
6. Positive
7. Invalid (No control)
8. Positive
9. Positive
10. Negative

Sample test #3

1. Invalid (No control)
2. Negative
3. Positive
4. Positive
5. Positive
6. Negative
7. Positive
8. Positive
9. Positive
10. Invalid (No control)